# INSTRUCTIONS

## eXact® Clear Orthodontic Adhesive

### Light Cure

#### Series 150-151

### ENGLISH

eXact® Light Cure Orthodontic Adhesive can be used to bond metal and ceramic brackets.

#### Clean Teeth

 Clean teeth thoroughly with plain pumice (no oils or fluoride) using a rotating bristle brush or rubber cup. Rinse teeth thoroughly with water and dry with air that is free of oil and moisture.

#### Condition Teeth

 Apply Enamel Conditioner (Order Number 150-020) on enamel surface according to directions enclosed in conditioner package. Rinse teeth thoroughly with water to remove all traces of conditioner and evacuate water. Maintain a dry field and do not allow patient to contaminate enamel surfaces.

 If conditioned areas do not appear chalky white, recondition for approximately 10 more seconds.

#### Light Cure Sealant

 Dispense one or two drops of light cure sealant onto a work pad. Using a disposable brush, apply a thin uniform coat of sealant on conditioned enamel surfaces.

 Note: Applying a thin layer of Light Cure Sealant on the bracket base can further improve bond strength.

#### Apply Paste

 Remove the Luer Loc Cap from the syringe. Place the recappable Luer Lock tip onto the syringe. Remove the cap and retain the cap for sealing the tip after bonding.

 Apply a small amount of eXact adhesive paste onto the bracket base.

 When bracket is positioned properly, press to seat firmly. Carefully remove excess paste from around bracket base.

#### Cure Paste

Curing lights vary in intensity. Some wand lights might require longer curing times.

**Ceramic Brackets.** Direct the curing light to focus directly through the labial surface of the bracket for 20 seconds by placing the light approximately 2 mm from the bracket.

**Metal and Plastic Brackets.** Direct the light to shine on the mesial and distal edges of the bracket for 15 seconds on each edge. Place the light approximately 5 mm from the bracket. Adjacent sides of two brackets can be cured at the same time.

#### Place Archwires

Archwires can be placed immediately after last bond is cured.

#### Remove Attachments

**Ceramic Brackets.** Remove with extreme care according to bracket manufacturer's instructions.

**Metal and Plastic Brackets.** Using a ligature cutter, deflect the bracket base and peel the bracket from the tooth. Most of the adhesive will adhere to the bracket base. Tooth surfaces may be further cleaned with a scaler or rubber wheel.

#### Storage

Best temperature for use is 68°-77°F (20°-25°C). Storage temperature is best at 35°-80°F (2°-27°C). eXact Light Cure Adhesive has a shelf life of three years.

CAUTION: eXact Light Cure Adhesive contains acrylate monomers which are known to produce allergic skin reaction or irritation to eyes on some individuals. Avoid contact with eyes, gingiva and skin. Wear gloves when handling. Immediately flush eyes with water, wash affected skin areas with soap and water, and get medical attention if irritation persists. Discontinue use if irritation occurs. Do not take internally.

#### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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© 2015 TP Orthodontics, Inc.
eXact is a registered trademark of TP Orthodontics, Inc.