Polymer Mesh Base Read-Base Brackets with eXact Pre-Applied Adhesive

The products covered by this instruction include:

- ClearVu® Mini Ceramic Brackets
- InVu® Ceramic Brackets
- Tip-Edge® PLUS Ceramic Brackets
- Hybrid Brackets and Tubes

ClearVu, InVu, eXact and Tip-Edge are registered trademarks of TP Orthodontics, Inc.

Products with polymer mesh base and pre-applied adhesive are manufactured under the following patents: US 5,263,859, 6,682,345, 6,685,468, 6,786,720, 6,834,781, 7,025,591, 7,131,826, 7,780,442, 8,029,275, 8,925,719, RE-45,743, Brazil BR1120150234003, Britain 1352617, 1545380, 1562511; China 1554190, 03806949.0, 03822189.6, ZL03824899.1; Europe 1352617, 1545380, 1562511, 2424461, 14773271.1; France 1545380, 1562511; Germany 1545380, 60315212.0, 603297861.2, 60332263.8-06; Germany/UK 2424461; Hong Kong 1166686; Italy 1352617, 1539012, 1545380, 1562511; Japan 4391242, 4414334, 4803940, 5415612; Mexico 265725, 269871, 274623, 3222623; Spain 1352617, 1539017, 1562511, ES2362089T3, ES2362089TE; Switzerland 1352617. All other patents pending.

ENGLISH

Bonding

Enamel Preparation

Please note: **Do not use adhesion promoters or enhancers on enamel.**

Clean teeth thoroughly with plain pumice (no oils or fluoride) using a rotating bristle brush or rubber cup. Rinse teeth thoroughly with water, isolate and dry.

Condition teeth by dispensing two to three drops of enamel conditioner onto work pad. Using one disposable foam pellet, dab conditioner on teeth; do not rub. Condition teeth for 20 seconds.

Rinse teeth thoroughly with water to remove all traces of conditioner and evacuate water. Maintain a dry field. Do not allow patient to contaminate enamel surfaces.

If conditioned areas do not appear chalky white, recondition for approximately 10 seconds.

When bonding to gold, amalgam, stainless steel, acrylic or composite restorations, use an adhesion promoter.

Apply Sealant Resin

Dispense two or three drops of sealant resin onto work pad. Using a disposable brush, apply a THIN, UNIFORM coat of sealant on conditioned enamel surfaces. DO NOT CURE SEALANT AT THIS TIME.

Place Brackets

Important Note: Each bracket is sealed in its own light-protective foil-sealed pod. **Note that the shelf life will be compromised once a pod is opened and resealed. To prevent adhesive from premature curing, do not open pod until ready for bracket placement.**

After determining tooth to be bonded, peel back foil lid of the individual bracket pod. The bracket is oriented in the pod to facilitate placement. Use bracket placing forceps or tweezers to grasp the bracket on the mesial/distal side surfaces and lift the bracket out of the pod. Brackets equipped with long axis indicators or positioners must be grasped by the axis indicator or positioner to lift out of the pod. Press bracket onto appropriate tooth with firm pressure. Remove any excess adhesive from around the bracket base and cure.

Cure Adhesive

Using a high intensity blue light curing unit, focus directly through the labial surface of the bracket for approximately 20-30 seconds.

Curing lights may vary in intensity. Some wand lights may require longer exposure times. Refer to manufacturer's instructions for use recommendations and required eye protection when using a curing light.

Note for brackets equipped with axis indicators or positioners: When the bracket has been completely cured with light, remove the long-axis indicator from the bracket by grasping with tweezers or forceps. Pull up and off away from the bottom of the bracket. Proceed with normal archwire placement.

Positional Markings

Aesthetic brackets may include temporary markings within the horizontal and vertical slots that serve as positional guides during bonding. These markings may be removed after bonding, and prior to installing the archwire, by cleaning with a standard toothbrush, toothpaste and water.

Rebonding

In the case of a premature debond, do not rebond. The polymer mesh base would become damaged when debonding and would not bond to the enamel properly if reused.

Clean excess adhesive from tooth, prepare the tooth surface according to the adhesive manufacturer's instructions, and bond a new bracket.

Debonding

Polymer mesh bases debond without the need for special instruments or techniques. The polymer mesh base allows safe and easy debonding. The debonding instrument of choice is a ligature cutting plier. Squeeze mesiodistal edges of the mesh base, grasping as closely as possible to the enamel surface. Do not grasp the bracket itself. The polymer mesh base will easily release from the tooth, while retaining most of the adhesive.

As an added safety feature, it is very helpful to leave brackets ligated to the archwire when debonding. If excessive bonding adhesive prevents applying pressure to the base or tends to deflect the backs of the ligature cutting plier away from the polymer mesh base, remove the adhesive from the mesiodistal surfaces of the base with a steel fissure bur. Avoid using a carbide bur or diamond instrument as they are not necessary and could accidentally mar the enamel surface. Once the adhesive is removed, apply compressive pressure directly to the edges of the bracket base near the enamel.

Storage and Use

Store brackets at temperatures between 35°F and 80°F (2°C and 27°C).

Protect from fluctuations in temperature. Bring to room temperature before use.

Read-Base Pre-Applied Adhesive brackets have an 18-month shelf life.

CAUTION: BRACKETS ARE EXTREMELY LIGHT SENSITIVE!

Do not expose uncured Read-Base brackets to ambient light. When not in use, store brackets in original packaging at room temperature. The polymer mesh base is prepared with a light cured adhesive. The adhesive is light-activated and can begin to set if exposed to strong artificial or natural light.

General Precautions

WARNING: Adhesive components are known to produce allergic skin reactions in certain sensitive individuals. Discontinue use if irritation occurs.

Wear gloves when handling adhesive materials. The natural oils in skin can compromise the properties of the bonding base. Avoid contact with eyes, gingival tissue and skin. Avoid ingesting.

If adhesive comes into contact with eyes, flush immediately with water. If adhesive comes into contact with skin, wash affected area with soap and water. If irritation persists, follow up with medical attention.

Special Care for Ceramics

- Do not touch polymer mesh bases prior to bonding.
- Ceramic is such a hard material that abrasion of the enamel can occur if incisal surfaces occlude. Each patient's occlusion should be taken into consideration when determining if and when ceramics can be safely bonded and still avoid contact with opposing teeth.
- Remove any stray adhesive from bracket surface and markings prior to curing. Cured adhesive covering any bracket identification or positional marking will prevent removal of these markings.
- Do not force auxiliaires or archwires into ceramic.
- Steel ligatures should not be tied as tightly on ceramic as on stainless steel.

Note: This is a single-use device. Once removed from the initial patient, it should be discarded in a proper manner. Any reuse of this product will run the risk of cross-contamination. TPO® expressly disclaims any liability caused by reuse.

NOTICE: TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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