# INSTRUCTIONS

## Polymer Mesh Base Products

Series 285, 297, 298, 299 and 391

### Polymer Mesh Base

The products covered by this instruction include:

- InVu® Aesthetic Brackets
- ClearVu® Aesthetic Brackets
- Click-It® Self-Ligating Brackets
- Tip-Edge PLUS® Ceramic Brackets
- MXi® Ceramic Brackets
- Hybrid Brackets and Tubes

### ENGLISH

*This is a single use product and once removed from the initial patient should be discarded in a proper manner. TPO expressly disclaims any liability caused by reuse.*

#### Bonding

Please note: **Do not use adhesion promoters or enhancers on enamel.**

No special bonding technique or bonding system is required for polymer mesh bases. Follow the adhesive manufacturer's instructions for a traditional mechanical bond, just like metal brackets. The patented micro polymer mesh base has undercuts that enable a mechanical interlocking to occur.

Polymer mesh bases can be safely bonded and debonded from crowns and other non-enamel surfaces. No special conditioners or primers are required. When bonding to gold, amalgam, stainless steel, acrylic or composite restorations, use of an adhesion promoter is acceptable.

#### Positional Markings

Aesthetic brackets may include temporary markings within the horizontal and vertical slots that serve as positional guides during bonding. These markings may be removed after bonding, and prior to installing the archwire, by cleaning with a standard toothbrush, toothpaste and water.

#### Rebonding

In the case of a premature debond, do not rebond. The polymer mesh base would become damaged when debonding and would not bond to the enamel properly if reused.

Clean excess adhesive from tooth, prepare the tooth surface according to the adhesive manufacturer's instructions, and bond a new bracket or tube.

#### Debonding

Polymer mesh bases debond without the need for special instruments or techniques. The polymer mesh safety base allows safe and easy debonding. The debonding instrument of choice is a ligature cutting plier. Squeeze the mesiodistal edges of the mesh base, grasping as closely as possible to the enamel surface. Do not grasp the bracket itself. The polymer mesh base will easily release from the tooth while retaining most of the adhesive.

As an added safety feature, it is very helpful to leave brackets, tubes, etc. ligated to the archwire when debonding. If excessive bonding adhesive prevents applying pressure to the base or tends to deflect the beaks of the ligature cutting pliers away from the polymer mesh base, remove the adhesive from the mesiodistal surfaces of the base with a steel, fissure bur. Avoid using a carbide bur or diamond instrument as they are not necessary and could accidentally mar the enamel surface. Once the adhesive is removed, apply compressive pressure directly to the edges of the base near the enamel.

#### Special Care for Ceramics

- Do not touch polymer mesh bases prior to bonding.
- Ceramic is such a hard material that abrasion of the enamel can occur if incisal surfaces occlude. Each patient's occlusion should be taken into consideration when determining if and when ceramics can be safely bonded and still avoid contact with opposing teeth.
- Remove any stray adhesive from bracket surface and markings prior to curing. Cured adhesive covering any bracket identification or positional marking will prevent removal of these markings.
- Do not force auxiliaries or archwires into ceramic.
- Steel ligatures should not be tied as tightly on ceramic as on stainless steel.

Note: This is a single-use device. Any reuse of this product will run the risk of cross-contamination to patients.

#### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

**Order online at [tportho.com](https://www.tportho.com).**

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ClearVu, Click-It, InVu, MXi and Tip-Edge PLUS are registered trademarks of TP Orthodontics, Inc.