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TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672 USA

TPortho.com

Auth. EU Rep.
TP Orthodontics Europe
Fountain Ct.
12 Bruntcliffe Way
Morley, Leeds LS27 0JG UK

ENGLISH

To access ligatures from one of the outside compartments:

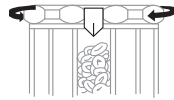
Figure A.

1. Rotate cap clockwise until vertical pointer on side of cap points to desired color (Figure A).
2. When viewed from top, the two outer openings will coincide forming a star over color selected (Figure B).

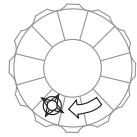
Figure B.

3. Shake dispenser like a salt shaker to release ligatures.
4. If desired, turn cap counterclockwise 10° to 15° to close compartment after dispensing.

To access ligatures from the large, center compartment:

Figure C.

1. Rotate cap counterclockwise as indicated by small arrow until two inner circles meet and form a star (Figure C).

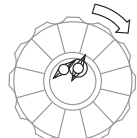
Figure D.

2. Shake the dispenser like a salt shaker to release the ligatures.
3. After dispensing, cap can be reversed (Figure D) 10° or 15° to close compartment. (Inner star will disappear.)

Note: Cap should not be removed for dispensing or loading ligatures. If for some reason the cap is removed, be sure the control disc underneath the cap is replaced with the four lugs facing downward.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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