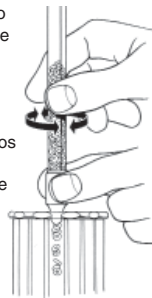
## PORTUGUESE

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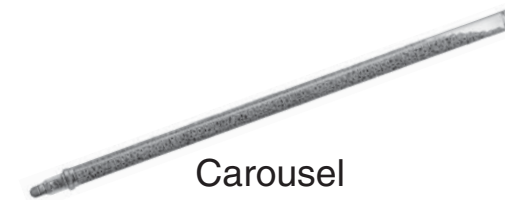
**Order online at [tportho.com](http://tportho.com).**

# INSTRUCTIONS

**Carousel  
Ligature Dispenser**

**Refill Tube**  
Series 361 and 382


TP Orthodontics, Inc.  
100 Center Plaza  
La Porte, Indiana 46350-9672 USA

[tportho.com](http://tportho.com)

## ENGLISH

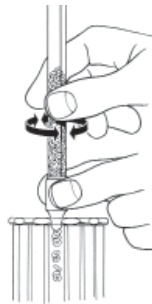1. Rotate Carousel Ligature Dispenser cap clockwise until vertical pointer on side of cap points to compartment to be refilled.
2. Remove plug from dispensing tip of the refill tube.
3. Place refill tube in an upright position directly over open hole on the top of dispenser.
4. Grasp the dispensing tip between the index finger and thumb of one hand.
5. Grasp refill tube with other hand and twist tube back and forth rapidly.
6. Continue twisting until the compartment is filled  $\frac{1}{4}$ " from the top. *Note: Overfilling prevents ligatures from dispensing when shaken.*
7. Slowly tilt refill tube away from the carousel and return plug to the top of dispensing tip.

**Note: Do not remove carousel cap to refill compartments.**

*Each refill tube holds approximately 2,000 elastomeric ligatures. The outer compartments of the Carousel Ligature Dispenser can be refilled nearly six times from one refill tube.*

### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

Order online at [tportho.com](http://tportho.com).

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