PORTUGUESE

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ENGLISH

Color-Coded Torque Bars

Torque Bars are now color-coded to help distinguish the correct orientation when placed in the patient's mouth.

30° Bars are marked red on the labial surface
20° Bars are marked blue on the labial surface

Upper Lingual Root Torque — marking is on patient's right
Lower Lingual Root Torque — marking is on patient's left

Note: This is a single-use device. Any reuse of this product will run the risk of cross-contamination to patients.

Warning: This product contains nickel. Direct patient to consult physician if allergic reaction occurs.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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