*CrossLink® Cement Intensifier is intended for use with CrossLink Light Cure Cement. When used in combination with CrossLink Cement, the Intensifier increases bond strength to metal band surfaces and is essentially moisture or saliva insensitive.*

## Instructions

1. CrossLink Cement Intensifier is applied on the inner band surface just prior to the application of CrossLink Cement.
2. Dispense a few drops of Intensifier on a mixing pad and apply to the inside of the band with a small bristle brush.
3. Apply a small bead of CrossLink Cement to the inside of the band. Spread the paste with a small spatula to evenly coat the entire inner band surface.

**Storage:** Best temperature for use is 68°-77° F (20°-25° C). Storage temperature is best at 35°-80° F (2°-27° C). CrossLink Cement Intensifier has a 3-year shelf life without refrigeration.

### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

Order online at [www.tportho.com](http://www.tportho.com).

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