INSTRUCTIONS

Polymer Mesh Base Products

Series 285, 297, 298, 299 and 391

Polymer Mesh Base

The products covered by this instruction include:

- InVu® Aesthetic Brackets
- ClearVu® Aesthetic Brackets
- Click-It® Self-Ligating Brackets
- Tip-Edge PLUS® Ceramic Brackets
- MXi® Ceramic Brackets
- NV® Hybrid Brackets
- NV® Hybrid Molar Tubes

InVu, ClearVu, Click-It, Tip-Edge PLUS, MXi and NV are registered trademarks of TP Orthodontics, Inc. InVu is manufactured under US Patents 6,685,468, 6,746,242, 7,780,442; Britain 1562511; China 03822189.6, ZL03824889.1; Europe 1545380; France 1562511; Germany 60332263.8-08; Italy 1562511; Japan 4414334, 4503440; Mexico 265725, 274623; Spain 1562511, ES2362089T3. ClearVu is manufactured under US Patents 6,746,242, 6,786,720, 7,131,836, 7,780,442; China ZL03824889.1; Europe 1352617, 1562511; Germany 60315212.0, 60332263.8-08; Japan 4503440; Mexico 274623. Click-It is manufactured under US Patent 7,025,591, 8,029,275; Japan 5415612. Tip-Edge PLUS is manufactured under US Patent 6,682,345; Britain 1539017; China 03806949.0; France 1539017; Germany 60327861.2; Italy 1539017; Japan 4391242; Mexico 269871; Spain 1539017. MXi is manufactured under US Patent 5,263,859, 6,786,720, 6,685,468, 7,780,442; Britain 1352617; China 03822189.6; Europe 1545380; Germany 60315212.0; Italy 1352617; Japan 3,334,940, 4414334; Mexico 265725; Spain 1352617, ES2362089T3; Switzerland 1352617. NV is manufactured under US Patents 5,263,859, 6,685,468, 6,746,242, 7,780,442; Britain 1545380, 1562511; China 03822189.6, ZL 03824889.1; Europe 1545380, 1562511; France 1545380, 1562511; Germany 1545380, 60332263.8-08; Italy 1545380, 1562511; Japan 4414334, 4503440; Mexico 265725, 274623; Spain ES2362089TE, 1562511. All other patents pending.

 TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672 USA

tportho.com

 0120 TP Orthodontics Europe
Fountain Ct.
12 Bruntcliffe Way
Morley, Leeds LS27 0JG UK

ENGLISH

This is a single use product and once removed from the initial patient should be discarded in a proper manner. TPO expressly disclaims any liability caused by reuse.

Bonding

Please note: **Do not use adhesion promoters or enhancers on enamel.**

No special bonding technique or bonding system is required for polymer mesh bases. Follow the adhesive manufacturer's instructions for a traditional mechanical bond, just like metal brackets. The patented micro polymer mesh base has undercuts that enable a mechanical interlocking to occur.

Polymer mesh bases can be safely bonded and debonded from crowns and other non-enamel surfaces. No special conditioners or primers are required. When bonding to gold, amalgam, stainless steel, acrylic or composite restorations, use of an adhesion promoter is acceptable.

Rebonding

In the case of a premature debond, do not rebond. The polymer mesh base would become damaged when debonding and would not bond to the enamel properly if reused.

Clean excess adhesive from tooth, prepare the tooth surface according to the adhesive manufacturer's instructions, and bond a new bracket or tube.

Debonding

Polymer mesh bases debond without the need for special instruments or techniques. The polymer mesh safety base allows safe and easy debonding. The debonding instrument of choice is a ligature cutting plier. Squeeze the mesiodistal edges of the mesh base, grasping as closely as possible to the enamel surface. Do not grasp the bracket itself. The polymer mesh base will easily release from the tooth while retaining most of the adhesive.

As an added safety feature, it is very helpful to leave brackets, tubes, etc. ligated to the archwire when debonding. If excessive bonding adhesive prevents applying pressure to the base or tends to deflect the beaks of the ligature cutting pliers away from the polymer mesh base, remove the adhesive from the mesiodistal surfaces of the base with a steel, fissure bur. Avoid using a carbide bur or diamond instrument as they are not necessary and could accidentally mar the enamel surface. Once the adhesive is removed, apply compressive pressure directly to the edges of the base near the enamel.

Special Care for Ceramics

- Do not touch polymer mesh bases prior to bonding.
- Ceramic is such a hard material that abrasion of the enamel can occur if incisal surfaces occlude. Each patient's occlusion should be taken into consideration when determining if and when ceramics can be safely bonded and still avoid contact with opposing teeth.
- Do not force auxiliaries or archwires into ceramic.
- Steel ligatures should not be tied as tightly on ceramic as on stainless steel.

Note: This is a single-use device. Any reuse of this product will run the risk of cross-contamination to patients.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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