Bonding Instructions

TPO's Porcelain Primer is used to prepare the surface of glazed dental porcelain for bonding orthodontic attachments. The silane coupling agent in the primer chemically unites the acrylic bonding adhesive and the silicon in porcelain.

- Using rubber cup or bristle brush with pumice, clean the porcelain along with the other teeth in the arch. Rinse teeth thoroughly with water and dry.
- Apply the porcelain primer to the porcelain surface with a cotton or foam pellet. Allow the primer to remain, undisturbed, on the porcelain surface for 2 minutes. Do not rinse to remove primer.
- The porcelain surface is now prepared for bonding. If using TPO's 1-to-1[®] Bonding System, a stronger bond is attained on the porcelain by eliminating the application of sealant.
- In most cases, the above procedures will provide a secure bond of attachment to porcelain. If the bond fails, it will be necessary to break the glaze on the porcelain with a small stone and repeat Steps 1 through 3.

CAUTION: Do not take primer internally. Avoid contact with eyes and soft tissue. In case of contact, flush immediately with water and get medical attention for eyes. Solution is flammable. Handle with care.

Removal of Attachments

Apply de-bonding forces carefully to avoid damage to porcelain. Attachments can be removed from a single wing bracket with a pin and ligature cutting plier by a "peeling" action. Twin brackets can be removed with a serrated beak plier by squeezing the wings together. Remove the remaining adhesive with rubber or abrasive wheels or cups. Polish with pumice.

Storage: Best temperature for use is 68°-77° F (20°-25°C). Storage temperature is best at 35°-80° F (2°-27° C). Porcelain Primer has a 3-year shelf life.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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