Características Gerais

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Reverse Pull Headgear Instructions

General Features

Reverse Pull Headgear is entirely made from stainless steel wire and non-toxic acrylic. It delivers anteriorly directed force to intraoral sites while avoiding intraoral anchorage. Two screws for anchorage of elastics are made of stainless steel and are mounted on the headgear.

Wire attachments for both chin cup and forehead rest are adaptable to perform all size adjustments. This can be accomplished by using the set screws and wrench supplied that allow the bar to slide into the attachments. Attachments on the traction bar are also adaptable and locked by set screws in metal housing to permit the postero-anterior movement necessary for treatment. Forehead and chin pads are in natural leather for better patient comfort.

Treatment Hints

Reverse Pull Headgear is intended for treatment of maxillary insufficiencies, mandibular prognathism and cleft palate. The reverse orthopedic pull occurring between maxilla and mandible facilitates the orthodontic treatment afterwards. The use of Reverse Pull Headgear is intended for modifying growth of bones rather than moving teeth. It stimulates rotation of the maxilla on the upper cranial implantation of the dento-alveolar arches in its entirety.

Extra protective caps enclosed and should be placed after adjusting appliance.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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