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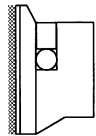
Order online at [www.tportho.com](http://www.tportho.com).

Mfg. by  
TP Orthodontics, Inc.

100 Center Plaza  
La Porte, Indiana 46350-9672 USA  
[www.tportho.com](http://www.tportho.com)

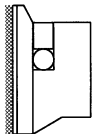
Auth. EU Rep.  
TP Orthodontics Europe  
Fountain Ct., 12 Bruntcliffe Way  
Morley, Leeds LS27 0JG UK

# Ribbon Torque Auxiliary Instructions

U.S. Patent # 4,676,747.

## Suggested Installation

For ease of installation and to prevent extrusion of the central and lateral incisors, the nickel-titanium torquing auxiliary is placed gingival to the main .020" round archwire.

First the main arch is seated in the base of the bracket slots. The torquing auxiliary is then placed over the main wire in one of the central brackets. Both are pinned in place with a hook pin.

The ends of the torquing auxiliary are then brought down into the lateral bracket slots and held in place with a spring-pin or hook pin.

A hook pin is then used to secure both the main archwire and auxiliary in the remaining central incisor bracket.

### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

Order online at [www.tportho.com](http://www.tportho.com).

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