# INSTRUCTIONS

## SAL<sup>®</sup> Retainer System

Series 392

Made by  
TP Orthodontics, Inc.  
100 Center Plaza  
La Porte, Indiana 46350-9672 USA

TPOortho.com

Auth. EU Rep.  
TP Orthodontics Europe  
Fountain Ct.  
12 Bruntcliffe Way  
Morley, Leeds LS27 0JG UK

SAL is a registered trademark of TP Orthodontics, Inc. and manufactured under US Patent 6,135,767; Brazil PI0012557-1; Britain 1204383; Germany 1204383; Italy 1204383; Japan 4554859; Mexico 253037.  
© 2003 TP Orthodontics, Inc. 996-226, Rev. 6/14

### ENGLISH

**NOTE:** Make all adjustments or modifications on the stainless steel portion of SAL loops. Do **not** grasp or bend the SAL connecting tube while making adjustments. Do **not** attempt to rotate wires in the SAL connecting tubes.

#### Step 1.


The retainer or spring aligner is first fabricated completely except for the labial acrylic segment. Place the nickel titanium SAL loops in their unactivated positions.

Step 1

#### Step 2.

The nickel titanium loops are then activated using a steel or nylon ligature to draw the ends toward each other. The degree of activation can easily be adjusted by how far the loops are drawn toward each other. Care should be taken not to activate the retainer excessively, since this may cause the retainer to spring up after being seated. If maximum activation is desired, use posterior clasps to aid in retention of the appliance.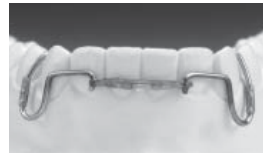

Step 2

#### Step 3.

Retainer fabrication is completed by placing final acrylic over the ends of the nickel titanium loops and across the labial surfaces of the anterior teeth. Once the retainer is delivered, it is recommended that the patient be seen one month later to ensure that it is being worn properly and functioning as desired. After this, the patient can be dismissed for much longer periods between retainer checks, since tightening of the retainer is not required.

Step 3

#### TPO's Custom Retainer Service

TP Orthodontics has been providing highest quality prescription appliances for over 60 years. Custom retainers, spring aligners and functionals are made to exacting standards by carefully trained technicians. Let TPO fabricate the new SAL retainer system in your next appliance. Call TPO for prescription forms and mailing supplies.

NOTE: This is a single-use device. Any reuse of this product will run the risk of cross-contamination to patients.

#### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

Order online at [www.tportho.com](http://www.tportho.com).

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