PORTUGUESE

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tportho.com

ENGLISH

These directions are written so that the patient (or a parent) can adjust the expander at home. Your orthodontist has provided you with a schedule for adjusting the appliance. Please follow these instructions carefully for the best possible treatment outcome.

Directions for Use of the Swivel Key

1. While holding the key handle, adjust the wire so that it is pointing straight forward.

2. Turn the handle so that the wire is facing upward. The button hinge should be facing either to the right or left side.

3. Carefully insert the end of the wire into the hole in the screw mechanism.

4. Push key gently but firmly toward the back of the mouth. The wire will tilt as the button hinge rotates. Continue to push until the next hole appears.

5. Stop pushing on the key and allow the wire to drop out of the hole in the screw mechanism. Do NOT pull the key back and forth.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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