P O R T U G U E S E

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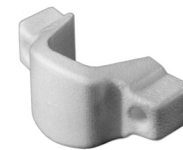
Order online at www.tportho.com.

Disposable Tongue-Away® Shield Instructions

Tongue-Away is a registered trademark of TP Orthodontics, Inc.

 Mfg. by
TP Orthodontics, Inc.
100 Center Plaza
La Porte, IN 46350-9672 USA
www.tportho.com

© 2006 TP Orthodontics, Inc.
996-583, Rev. 5/07

ENGLISH

Tongue-Away® Shields are designed for use during direct bonding of orthodontic attachments. The bite blocks allow the patient to relax. The shield prevents the tongue from contaminating the bonding surfaces.

The holes in the bite blocks will accept a saliva evacuation tube if desired and may be used for grasping with pliers during placement and removal.

CAUTION: It is recommended that a safety string be attached through hole in one of the bite blocks.

Disposable Tongue-Away Shields are not meant to be sterilized or used on more than one patient. Dispose of shield after each patient.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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