
ENGLISH

To access ligatures from one of the outside compartments: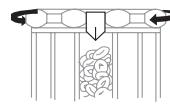

1. Rotate cap clockwise until vertical pointer on side of cap points to desired color (Figure A).
2. When viewed from top, the two outer openings will coincide forming a star over color selected (Figure B).

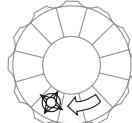
Figure B.

To access ligatures from the large, center compartment:

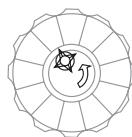
Figure C.

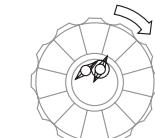
Figure D.

Note: Cap should not be removed for dispensing or loading ligatures. If for some reason the cap is removed, be sure the control disc underneath the cap is replaced with the four lugs facing downward.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, any of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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© 2006 TP Orthodontics, Inc.