ENGLISH

Size Selection


Using Special Millimeter Ruler

Ruler is designed to help select the proper size Bondable Lingual Retainer by measuring either six, four, three or two anterior teeth. Ruler may be used on patient's right or left side, according to operator's preference.

Measuring for Lateral to Lateral Retainer

Using ruler with green numerals, place the point distal to either the lower right or left lateral, bend the ruler along the incisolingual edges and take reading distal to opposite lateral. Select lingual retainer having the same size measurement.

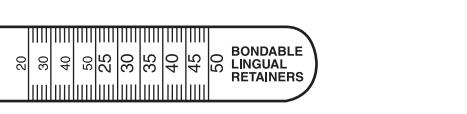Combination Cuspid-Bicuspid and Bicuspid to Bicuspid Retainers

Using ruler with red numerals, a reading may be obtained in one of two ways:

1. For quick, easy measurement in the mouth, use the condensed scale of the ruler with the small numerals. Place the point distal to either the right or left lower cuspid (right or left lower bicuspid if measuring for bicuspids) curve the ruler along the incisolingual edges. Reading at midline indicates total mesiodistal width from cuspid to cuspid or bicuspid to bicuspid.
2. For more accurate measurement in the mouth, or when measuring a model, use the full-length scale with the large numerals. Place the point distal to either the lower right or left cuspid, (right or left lower bicuspid if measuring for bicuspid to bicuspid retainer) bend the ruler along the incisolingual edges, and take reading distal to opposite cuspid (or bicuspid).

Measuring for Cuspid to Cuspid

Combination Cuspid-Bicuspid and Bicuspid to Bicuspid Retainers

Using ruler with red numerals, a reading may be obtained in one of two ways:

1. For quick, easy measurement in the mouth, use the condensed scale of the ruler with the small numerals. Place the point distal to either the right or left lower cuspid (right or left lower bicuspid if measuring for bicuspids) curve the ruler along the incisolingual edges. Reading at midline indicates total mesiodistal width from cuspid to cuspid or bicuspid to bicuspid.
2. For more accurate measurement in the mouth, or when measuring a model, use the full-length scale with the large numerals. Place the point distal to either the lower right or left cuspid, (right or left lower bicuspid if measuring for bicuspid to bicuspid retainer) bend the ruler along the incisolingual edges, and take reading distal to opposite cuspid (or bicuspid).

incisolingual edges. Reading at midline indicates total mesiodistal width from cuspid to cuspid or bicuspid to bicuspid.

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Note: Each mark on the condensed scale equals two millimeters.

2. For more accurate measurement in the mouth, or when measuring a model, use the full-length scale with the large numerals. Place the point distal to either the lower right or left cuspid, (right or left lower bicuspid if measuring for bicuspid to bicuspid retainer) bend the ruler along the incisolingual edges, and take reading distal to opposite cuspid (or bicuspid).

Selecting Lingual Retainer

Select lingual retainer from the proper series having the same size measurement.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability of TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with user, or inability to use, TPO's products.

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Measuring for Cuspid to Cuspid (lingual retentieapparaat)

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