## ENGLISH

Tongue-Away Shields are designed for use during direct bonding of orthodontic attachments. The bite blocks allow the patient to relax. The shield prevents the tongue from contaminating the bonding surfaces.

Tongue-Away Shields are designed with three shield heights. Always select the shield size to afford the patient maximum comfort.

The holes in the bite blocks will accept a saliva evacuation tube if desired and may be used for grasping with a plier during placement and removal.

**CAUTION:** It is recommended that a safety string be attached through hole in one of the bite blocks.

Tongue-Away Shields are phthalate-free and can be cold-sterilized. Do not boil or autoclave. Rinse thoroughly to clear all sterilizing solution from holes and contours.

### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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© 2008 TP Orthodontics, Inc.
Tongue-Away is a registered trademark of TP Orthodontics, Inc.
Advena Ltd.