INSTRUCTIONS

TP Original Wire Wire Bending

Series 230-233

Symbols Used on Labeling

	Manufacturer.
	Date of manufacture.
	Authorized Representative in the European Community.
	Batch code.
	Caution! See warnings and precautions.
	Consult instructions for use.
	Do not re-use.
	Do not use if package is damaged.
	Product is not sterilized.

TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672 USA

tportho.com

1434 Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatara, BKR 4013
Malta


ENGLISH

TP Original Wire

TP Original wire is manufactured in the United States under carefully controlled conditions to assure consistent quality. However, due to the extreme hardness of TP Original wire, special care is required in bending. Occasional breakage should be anticipated, but the results obtained will more than compensate for this slight inconvenience.

For best results, please follow these bending instructions:

1. **Do not** attempt to straighten wire by stripping between plier beaks.
 2. Hold pliers very lightly when bending wire. Do not squeeze or pull on wire. Plier must have smooth beaks. Carbide tips are suitable, but only if polished.
 3. Bend wire very slowly, pressing with thumb or forefinger. Do not rotate plier while bending.
- Loops and circles should be formed around the square beak and beaks should separate slightly.**
4. Never pinch the wire with the plier beaks—before or during bending.
 5. Do not scratch wire to locate bends.

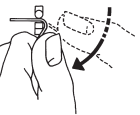TP Original wire becomes so hard from bending (work hardening) there is no need for heat treating and no margin to permit "back-bending" to correct mistakes.

Note: This wire may be easily broken if not properly handled.

Note: This is a single-use device. Any reuse of this product will run the risk of cross-contamination to patients.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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