INSTRUCTIONS

Straight Shooter®

Care and Use Instructions

Series 100

Symbols Used on Labeling

	Manufacturer.
	Date of manufacture.
	Authorized Representative in the European Community.
	Batch code.
	Caution! See warnings and precautions.
	Consult instructions for use.
	Do not use if package is damaged.
	Product is not sterilized.

ENGLISH

Straight Shooter® Ligature Gun

The TP Orthodontics Straight Shooter ligature gun is used to facilitate the placement of elastomeric ligatures.

The Straight Shooter is strictly designed for use with TPO elastomeric ligatures in Carousel Dispensers or mounted on Sticking® Ligature Holders. Do not use the Straight Shooter to remove ligatures from integrally molded dispensers, as damage to the wire fingertips may result.

Avoid ligatures that are smaller than .125" in diameter or have a Shore hardness that exceeds 80. Either of these factors will make it difficult, if not impossible, to engage the ligature over the archwire or the bracket.

TPO is not responsible for damage to the Straight Shooter when improper ligatures are used.

Loading Ligatures

It is possible to load a ligature onto the Straight Shooter in one of two ways: manually or automatically. To manually place a ligature, use your fingertips to fit it over the wire end tips of the Straight Shooter.

To use the automated loading method, place individual ligatures on a flattened surface, such as a Sticking® tabletopt. Place the wire fingertips into the center of a ligature. The ligature will fit tightly around the closed tips and remains there until applied to a bracket.

Positioning Ligatures

Move the loaded Straight Shooter toward the desired bracket while squeezing the handle slightly. This advances the plunger and separates the wire fingertips that begin expanding the ligature.

Bring the ligature into contact with the archwire and increase squeezing pressure on the trigger to further advance the plunger. The wire tips will spread farther apart. Make sure that the ligature can encircle both the gingival and incisal wings of the bracket. Continue moving the Straight Shooter toward the tooth until both the gingival and incisal wire fingertips are touching the crown.

Placing Ligatures

When the ligature is aligned in the proper position, further squeeze the trigger to fully extend the plunger. This will push the ligature off the wire fingertips and cause it to snap into place behind the wing tips of the bracket.

If the wire is labial to the bracket and relatively flexible (as might be the case with a nickel titanium, looped, or sectional arch), it can be cradled on the sides of the stretched ligature and carried toward the archwire slot. If excessive pressure is needed to carry the archwire onto the bracket, support the tooth lingually during ligation.

Troubleshooting the Straight Shooter

<i>Problem</i>	<i>Suggestion(s)</i>
Ligatures do not pick up on the end of wire fingers.	- Wrong size ligatures (inside diameters are either too large or small) - Bent finger(s) on Shooter - Weak spring or sticking plunger
Elastic ligature leaves the end of the Shooter prematurely.	- Excessive squeezing of handles before wire fingertips are aligned with tooth surface - One or more bent wires
Patient complains of excessive pressure during ligature placement.	- Ligatures are too small - Ligatures are not resilient enough; Shore hardness above 80 - Patient has a low pain threshold – support teeth on the lingual to counteract pressure

CAUTIONS: The Straight Shooter should not be used (either in practice or otherwise) unless everyone in the vicinity, including the patient, is wearing eyeglasses or protective eyewear. The elastomeric ligatures are "shot" off the end of the tip with high velocity and could cause severe eye damage.

Aiming the Straight Shooter away from those present is not sufficient to avoid this concern because it is possible for the ligature to exit the wire tips "sideways" — that is, not along a path in line with the barrel of the Straight Shooter. These varying paths of release are not significant during normal, planned use but can cause inconvenience, discomfort, and even harm if the stretched elastomeric ligature is released before there is firm contact with the archwire or if the wire fingers are not in contact with the tooth.

Considerations

The Straight Shooter ligature gun should be properly cleaned and sterilized after each patient use. Inspect the instrument before use. Do not attempt to use if wire fingers appear loose or broken, or if plastic body or handle appear cracked or damaged.

Cleaning/Sterilization

The life of the Straight Shooter ligature gun can be extended with the proper care. To clean, remove any extraneous material with gloved fingers. Wash with warm detergent and rinse thoroughly with water. The materials and construction of the Straight Shooter are chemically resistant to most cleaning media.

- Autoclave – An autoclave can be used to sterilize the Straight Shooter. Follow manufacturer's directions for proper use. Pre-vacuum stage is not required. Optional cycle time: 135°C (275°F) for six minutes.

- Heat Treatment – Heat treatment sterilization will not harm the Straight Shooter ligature gun so long as it has first been properly cleaned and dried. Follow manufacturer's instructions for proper technique, but do not exceed a temperature setting of 191°C (375°F) for periods greater than 30 minutes. Optional cycle time: 191°C (375°F) for 23 minutes.

- Cold Sterilization – If cold sterilizing, follow the directions suggested by the manufacturer of the sterilizing solution regarding concentration and time. Avoid solutions containing alkali, acidic (below pH 6.0) or highly reactive fluoridated compound which may cause damage to the Straight Shooter ligature gun.

Storage

Cleaned and sterilized Straight Shooter ligature guns may be stored together in a clean, dry container.

Disassembly

When disassembling Straight Shooter, protect eyes from injury as parts may be released under spring pressure.

- Grasp pin inside disassembly tool with Straight Shooter fingers.
- Open handles of Shooter and place end of disassembly tool on flat surface. Press down firmly. Push until shoulder of disassembly tool is flush with nozzle on front handle.
- Remove disassembly tool and remove plunger assembly keeping it intact.

Assembly

- Insert plunger assembly into rear of front handle. Apply light pressure on rear of plunger and rotate barrel until it snaps into place and barrel comes all the way forward.
- Place end of assembly tool without pin on rear of plunger assembly and press until assembly tool contacts handle.
- Remove assembly tool, fold handle down, and the Straight Shooter is ready for use.
- Wire fingers can be rotated 45° to suit your clinical preference. Grasp barrel, push toward handle, and rotate until barrel locks into position.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672 USA
tportho.com

Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

Straight Shooter and Sticking® are registered trademarks of TP Orthodontics, Inc. Straight Shooter is also registered in Mexico under trademark 326577.


ENGLISH

Ligatures are self-retaining around fingertips of Shooter.

Ligature begins to expand as handles are squeezed.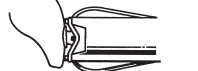

Ligature expanded to maximum size, plunger straddling bracket and fingertips touching the tooth.

Plunger advances pushing elastomeric ligature off fingertips around archwire and behind bracket wing tips.

Wire can be seated into slot during ligature placement.

Using ligatures that are too small will make placement impossible or uncomfortable.

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