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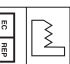
## ENGLISH

## Cleaning of Instruments (Prior to Sterilization)

For proper cleaning, no disassembly of the pliers is required. Do not rinse pliers in water. For best results, clean instruments prior to sterilization. DO NOT USE the following products for pre-sterilization cleaning:

- Soap and Water
- Iodophors (iodine)
- Softened water (Note that chemical solutions diluted with water may experience alteration of their chemical balance due to additives in the local water supply)
- Enzymatic cleaning solutions
- Acidic or strong alkaline cleaning solutions
- Cleaning solutions containing phosphates

## The protocol for cleaning instruments is as follows:

- Ultrasonically clean or hand scrub instruments in a biodegradable, no rinse solution. MP\_US multipurpose ultrasonic cleaning concentrate is a special formula containing a mild detergent, rust inhibitor and lanolin lubricant. Dilute 3 oz. or 90 mL of the solution with one gallon of water. Use distilled water for best results.
- Clean for five (5) minutes in a small capacity ultrasonic machine or twelve (12) minutes in a large capacity ultrasonic machine.
- Do not rinse after cleaning as Minerals present in tap water may leave rust spots on steel pliers.
- Dry pliers thoroughly by allowing them to drip dry or by patting dry with a clean towel. Removal of any remaining liquid from the joint areas (use of compressed air is an effective means of drying these areas).

## Lubrication

Lubricate hinged instruments with a silicone lubricant after cleaning and prior to sterilization. Do not use oil based lubricants. TPO recommends the use of Instrument Shield for lubrication and protection of instrument joints prior to heat sterilization.

## Sterilization

Dry heat sterilization is the recommended sterilization method for instruments. Steam sterilization (autoclaving) is not recommended as it is known to cause rust, pits and fissures in the steel instruments. Likewise, chemclaving (chemical under pressure) is not recommended as it may also lead to corrosion of pliers.

## Dry Heat Sterilization

TPO recommends dry heat as the most compatible method for destroying all living microorganisms. Inserted-tip instruments will have longer life if maintained without the use of moisture, pressure and chemicals.

For dry heat sterilization it is recommended that sterilization parameters are established per the manufacturer's instructions specific to the sterilizing equipment. Note that brown spots may occur where foreign material is not completely removed before dry heat sterilization, therefore it is recommended to follow the pre-sterilization cleaning instructions.

## Steam Sterilization (not recommended)

If autoclaving is the only option available, first clean by adding anti-rust tablets or solution to water source and milk bath to lubricate. Do not rinse. (If using a porous load autoclave or any other autoclave that creates a pre-vacuum, the pliers must also be properly wrapped before insertion into the autoclave.) It is recommended that the steam sterilization parameters be established per the manufacturer's instructions specific to the sterilizing equipment. Avoid any metal to metal contact of instruments. After steam sterilization, dry instruments thoroughly to remove any remaining liquid from joint areas.

## Immersion Disinfection/Sterilization

If using an immersion process to disinfect and/or sterilize instruments, please be aware of the following: Specific products within a general group of chemical compounds (such as 2% glutaraldehydes) may be formulated quite differently, and will therefore provide very different results in terms of rust and corrosion. Use only products containing rust inhibitors and those which have been demonstrated as safe for use on carbon steel. Procidie 28 Alkaline Glutaraldehyde without phenols is a recommended product for use in Immersion Disinfection/Sterilization.

- For best results, DO NOT immerse pliers in these materials:
  - Glutaraldehydes with added phenols
  - Iodophor (iodine) surface disinfectants
  - Chlorine dioxide compounds
  - Sodium hypochlorite (bleach) compounds
  - Water
  - Hydrogen peroxide
- Immerse instruments for the least amount of time possible. Follow the manufacturers guidelines of minimum exposure times for effectiveness.
- Avoid metal-to-metal contact of plier tips by opening jaws, loading with care and using plastic disinfection trays.
- After rinsing disinfectant from instruments, thoroughly dry tips and remove all moisture from joint areas.

## General Product Care

Instrument end of life is determined by wear and damage caused by use. To properly maintain pliers and extend their usable life, we recommend the following procedures and products for instrument care:

- Clean, lubricate and sterilize pliers as soon as it is practical after use.
- Store pliers in a clean, dry area.
- Lubricate pliers' hinge joints weekly with high-quality pure silicon lubricant.
- Periodically examine your instruments for any signs of unusual wear or discoloration:
  - Check hinge for smooth motion
  - Check for damage on blade inserts
  - Check for any signs of corrosion

## Warranty

TP Orthodontics, Inc. provides a one-year warranty from date of purchase on pliers. Defective pliers that have not been misused will be replaced at no charge during the warranty period. Please contact your local representative for return instructions.

## Sharpening and Repair Service

TPO recommends Precision Plier Service for out-of-warranty repairs and sharpening. Information can be found at *precisionplierservice.com*.

## NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for all loss, damages arising in connection with use, or inability to use, TPO's products.

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