INSTRUCTIONS

Norm-O® and Extend-O® Impression Trays Cleaning and Other Helpful Tips

Series 125

Symbols Used on Labeling

	Manufacturer.
	Date of manufacture.
	Authorized Representative in the European Community.
	Batch code.
	Caution! See warnings and precautions.
	Consult instructions for use.
	Use by date.
	Do not use if package is damaged.
	Product is not sterilized.

TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672 USA
tportho.com

Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

Extend-O and Norm-O are a registered trademarks of TP Orthodontics, Inc.
Teflon is a registered trademark of DuPont.

ENGLISH

Norm-O and Extend-O Impression Trays can be re-used numerous times, if properly cleaned and sterilized after each use. Inspect trays before using each time. Discard any trays that exhibit a blistered or peeling coating or loose or broken handles.

Cleaning

Preserve the life of your new impression trays by caring for them properly.

1. Remove bulk of alginate from tray with gloved fingers. Knives or metal spatulas will scratch the finish.
2. Place tray in cleaning solution recommended for Teflon® coated aluminum. Mix to the manufacturer's specifications.
3. Soak trays as specified in the cleaning solution Instructions.
4. Thoroughly rinse trays. Any remaining impression material should be rubbed off with gloved fingers.

Caution: If scraping is necessary, use a soft tooth brush or plastic spatula (i.e., Dispatula, 100-311) with minimal force to avoid scratching surface.

Sterilization

Impression trays can be autoclaved or heat sterilized.

Recommended guidelines for autoclaving is 120°C (248°F) to 137°C (278.6°F) for 17.5 minutes.

Dry heat sterilization is recommended up to 219°C (426.2°F) for 23 minutes. Do not exceed 230°C (446°F).

Trays will turn light brown where any saliva or residue has not been removed prior to heat treatment.

Avoid solutions containing alkaline, acidic (below pH 6.0) or highly reactive fluorinated compound which may damage tray.

Preparation

For patient comfort and to help hold impression material in place during removal from the mouth, place one strip of utility wax or Mor-Tight around periphery of each tray.

Storage

Cleaned and sterilized trays can be stored together in a clean, dry container.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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