INSTRUCTIONS

eXact® Clear Orthodontic Adhesive

Click the language you wish to view for eXact Adhesive to be taken directly to that page.

- English
- Dutch
- French
- German
- Italian
- Portuguese
- Spanish
- Chinese
- Japanese

TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672
USA

tportho.com

0297 Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

INSTRUCTIONS

ENGLISH

eXact® Clear Orthodontic Adhesive

Symbols Used on Labeling

	Manufacturer.
	Date of manufacture.
	Authorized Representative in the European Community.
	Batch code.
	Caution! See warnings and precautions.
	Consult instructions for use.
	Do not re-use.
	Use by date.
	Keep away from sunlight.
	Store between 2°C/35°F (lower limit) and 27°C/80°F (higher limit).
	Do not use if package is damaged.
	Medical device.
	Prescription device.

TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672
USA

tportho.com

0297 Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

eXact is a registered trademark of TP Orthodontics, Inc.

© 2024 TP Orthodontics, Inc.

996-194, Rev. D / 2.26

Indications for Use

eXact Clear Orthodontic Adhesives is indicated for use as an orthodontic bonding agent for metal brackets, ceramic brackets, and buccal tubes to the tooth's surface.

Warnings

- Best temperature for use is 68°-77°F (20°-25°C). Storage temperature is best at 35°-80°F (2°-27°C).
- Do not expose adhesive to heat or direct light prior to use as this can start polymerization of the adhesive.
- Do not use after expiration date.
- Ensure adhesive syringe is capped immediately after use.
- This product is designed and manufactured for single use.
- Phenolic substances (i.e., Eugenol, phenolic disinfectants) will inhibit polymerization. Do not let adhesive have contact or be stored near phenolic products as this may inhibit proper adhesive curing.

Precautions

eXact Clear Orthodontic Adhesive contains acrylate monomers which are known to produce allergic skin reaction or irritation to eyes on some individuals. Avoid contact with eyes, gingiva and skin. Wear gloves when handling. Immediately flush eyes with a large quantity of water, wash affected skin areas with soap and water, and get medical attention if irritation persists. Discontinue use if irritation occurs. Do not take internally.

Contraindications

- Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e. decalcification, carious lesions, large restorations) can increase the risk of tooth damage.
- TP Orthodontics sells its products to trained dental professionals and orthodontists, who have the primary responsibility to identify any possible contraindications that may preclude the use of this product.
- Federal law restricts this device to sale by or on the order of a dentist.

Instructions for Use

This product must be used by qualified orthodontists or dentists trained in orthodontics.

CLEAN TEETH

Clean teeth thoroughly with plain pumice (no oils or fluoride) using a rotating bristle brush or rubber cup. Rinse teeth thoroughly with water and dry with air that is free of oil and moisture.

CONDITION TEETH

Apply Enamel Conditioner on enamel surface according to manufacturer's directions. Rinse teeth thoroughly with water to remove all traces of conditioner and evacuate water. Maintain a dry field and do not allow patient to contaminate enamel surfaces.

If conditioned areas do not appear chalky white, recondition for approximately 10 more seconds.

LIGHT CURE SEALANT

Dispense one or two drops of light cure sealant onto a work pad. Using a disposable brush, apply a thin uniform coat of sealant on conditioned enamel surfaces.

Note: Applying a thin layer of Light Cure Sealant on the bracket base can further improve bond strength.

INSTRUCTIONS

eXact® Clear Orthodontic Adhesive


ENGLISH

APPLY PASTE

Note: Do not apply adhesive onto brackets until patient is ready to be bonded.

Apply a small amount of eXact adhesive paste onto the bracket base.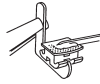

When the bracket is positioned, press to seat firmly. Carefully remove excess paste from around bracket base.

CURE PASTE

When using curing light with intensity of 300mW/cm² and average wavelength of 400-500nm, follow the instructions below. If your curing light does not meet these specifications, please refer to the light manufacturer's instructions for curing.

Ceramic Brackets. Direct the curing light to focus directly through the labial surface of the bracket for 20 seconds by placing the light approximately 2mm from the bracket.

Metal and Plastic Brackets. Direct the light to shine on the mesial and distal edges of the bracket for 15 seconds on each edge. Place the light approximately 5mm from the bracket. Adjacent sides of two brackets can be cured at the same time.

PLACE ARCHWIRES

Archwires can be placed immediately after last bond is cured.

REMOVE ATTACHMENTS

Ceramic Brackets. Remove with extreme care according to the bracket manufacturer's instructions.

Metal Brackets. No special debonding technique is required. Remove the bracket using a debonding plier or instrument of doctor's preference. Apply gentle pressure until the bracket debonds.

Storage Condition

Best temperature for use is 68°-77°F (20°-25°C). Storage temperature is best at 35°-80°F (2°-27°C).

Shelf Life

5 years.

REPORTING SERIOUS INCIDENTS

User and Patient Notice: If a serious incident occurs with this medical device, report it to the manufacturer and to the competent medical authority for the user/patient's country.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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tportho.com

0297

Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

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