# INSTRUCTIONS

## Orthodontic Brackets with Pre-Applied Adhesive

Click the language you wish to view for Orthodontic Brackets with Pre-Applied Adhesive to be taken directly to that page.

- English
- Dutch
- French
- German
- Italian
- Portuguese
- Spanish
- Chinese
- Japanese

**TP Orthodontics, Inc.**  
100 Center Plaza  
La Porte, Indiana 46350-9672  
USA

**tportho.com**

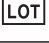
0297 Advena Ltd.  
Tower Business Centre, 2nd Flr.  
Tower Street, Swatar, BKR 4013 Malta

## Orthodontic Brackets with Pre-Applied Adhesive

The products covered by this instruction include:

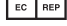
- ClearVu® with Pre-Coated Adhesive
- InVu® with Pre-Coated Adhesive
- Tip-Edge® PLUS Ceramic with Pre-Coated Adhesive
- Nu-Edge NX® with Pre-Applied Adhesive
- Tip-Edge NX with Pre-Applied Adhesive

### Symbols Used on Labeling

	Manufacturer.
	Date of manufacture.
	Authorized Representative in the European Community.
	Batch code.
	Caution! See warnings and precautions.
	Consult instructions for use.
	Do not re-use.
	Use by date.
	Keep away from sunlight.
	Store between 2°C/35°F (lower limit) and 27°C/80°F (higher limit).
	Do not use if package is damaged.
	Medical device.
	Prescription device.
	Contains hazardous substance.

 TP Orthodontics, Inc.  
100 Center Plaza  
La Porte, Indiana 46350-9672  
USA

[tportho.com](http://tportho.com)

 0297  Advena Ltd.  
Tower Business Centre, 2nd Flr.  
Tower Street, Swatar, BKR 4013 Malta

ClearVu, InVu, Nu-Edge NX, Tip-Edge, and TPO are registered trademarks of TP Orthodontics, Inc. Patents apply.

© 2024 TP Orthodontics, Inc.

996-427, Rev. C / 2.26

### Indications for Use

Intended for use in orthodontic appliance application and bonding for orthodontic treatment.

### Warnings

- Best temperature for use is 68°-77°F (20°-25°C). Storage temperature is best at 35°-80°F (2°-27°C). Do not expose adhesive to heat or direct light prior to use as this can start polymerizing the adhesive.
- Do not use after expiration date.
- Protect from fluctuations in temperature. Bring to room temperature before use. This product is designed and manufactured for single use.
- Do not expose uncured brackets to ambient light. When not in use, store brackets in original packaging at room temperature. The base is prepared with a light cured adhesive that can begin to set if exposed to artificial or natural light.
- This is a single-use device. Once removed from the initial patient, it should be discarded in a proper manner. Recycling of this product will run the risk of cross-contamination and may compromise material integrity, bond strength and product performance. TPO® expressly disclaims any liability caused by reuse.
- Do not open bracket pod until ready for appliance placement. Pod cannot be resealed and the shelf life will be compromised once a pod is opened.
- Do not use adhesion promoters or enhancers on enamel when bonding ceramic brackets.

### Precautions

- Pre-Applied Adhesive contains acrylate monomers which are known to produce allergic skin reaction or irritation to eyes on some individuals. Avoid contact with eyes, gingiva and skin. Wear gloves when handling. Immediately flush eyes with a large quantity of water, wash affected skin areas with soap and water, and get medical attention if irritation persists. Discontinue use if irritation occurs. Do not take internally.
- Metal brackets contain nickel and/or chromium, and a small percentage of the population is known to be allergic to these metals. In case of an allergic reaction, refer the patient to consult a physician.

### Contraindications

- Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.
- TP Orthodontics sells its products to trained dental professionals and orthodontists, who have the primary responsibility to identify any possible contraindications that may preclude the use of this product.
- Federal law restricts this device to sale by or on the order of a dentist.

### Instructions for Use

This product must be used by qualified orthodontists or dentists trained in orthodontics.

### BONDING

**Please note:** Do not use adhesion promoters or enhancers on enamel when bonding ceramic brackets.

- **Clean Teeth**  
Clean teeth thoroughly with plain pumice (no oils or fluoride) using a rotating bristle brush or rubber cup. Rinse teeth thoroughly with water and dry with air that is free of oil and moisture.
- **Acid Etching**  
Apply etching agent on tooth surface according to manufacturer's instructions. Rinse teeth thoroughly with water to remove all traces of conditioner and evacuate water. Maintain a dry field and do not allow patient to contaminate enamel surfaces.

If conditioned areas do not appear chalky white, recondition for approximately 10 more seconds.

## Orthodontic Brackets with Pre-Applied Adhesive

The products covered by this instruction include:

- ClearVu® with Pre-Coated Adhesive
- InVu® with Pre-Coated Adhesive
- Tip-Edge® PLUS Ceramic with Pre-Coated Adhesive
- Nu-Edge NX® with Pre-Applied Adhesive
- Tip-Edge NX with Pre-Applied Adhesive

**Note:** When bonding to gold, amalgam, stainless steel, acrylic or composite restorations, prepare the surface accordingly and use of an adhesion promoter is acceptable. When bonding to a porcelain surface, follow the Porcelain Conditioner instructions to prepare the bonding surface.

### • Sealant Application

Dispense one or two drops of light cure sealant onto a work pad. Using a disposable brush, apply a thin uniform coat of sealant on conditioned enamel surfaces. **DO NOT CURE SEALANT AT THIS TIME.**

### • Place Brackets

**Important Note:** Each bracket is sealed in its own light-protective foil-sealed pod. To prevent adhesive from premature curing, do not open pod until ready for bracket placement.

After determining tooth to be bonded, peel back foil lid of the individual bracket pod. The bracket is oriented in the pod to facilitate placement. Use bracket placing forceps or tweezers to grasp the bracket on the mesial/distal side surfaces and lift the bracket out of the pod. Brackets equipped with long axis indicators or positioners must be grasped by the axis indicator or positioner to lift out of the pod. Lightly place the bracket onto appropriate tooth, adjust final position and press firmly. Remove any excess adhesive from around the bracket base and cure.

### • Cure Adhesive

When using curing light with intensity of 300mW/cm<sup>2</sup> and average wavelength of 400-500nm, follow the instructions below. If your curing light does not meet these specifications, please refer to the light manufacturer's instructions for curing.

<b>Ceramic Brackets</b>	Direct the curing light directly through the labial surface of the bracket for 20-30 seconds or according to curing light manufacturer's recommendation.
<b>Metal Brackets</b>	Direct the curing light on the mesial and distal edges of the bracket for 15 seconds on each edge or according to curing light manufacturer's recommendation. Adjacent sides of two brackets can be cured at the same time.

Note for brackets equipped with axis indicators or positioning jigs: When the bracket has been completely cured with light, remove the long-axis indicator from the bracket by grasping with tweezers or forceps. Pull up and off away from the bottom of the bracket. Proceed with normal archwire placement.

### **Positional Markings**

Aesthetic brackets may include temporary markings within the horizontal and vertical slots that serve as positional guides during bonding. These markings may be removed after bonding, and prior to installing the archwire, by cleaning with a standard toothbrush, toothpaste and water.

## DEBONDING

Ensure there is no adhesive flash over the bracket base. Remove any adhesive flash prior to debond.

Care should be taken to contain small parts so they are not accidentally inhaled or ingested by the patient. As an added safety measure, it is recommended to leave brackets ligated to the archwire when debonding.

### • Ceramic Brackets

The debonding instrument of choice is a ligature cutting plier. Squeeze mesiodistal edges of the mesh base, grasping as closely as possible to the enamel surface. Do not grasp the bracket itself. The polymer mesh base will easily release from the tooth, while retaining most of the adhesive.


# INSTRUCTIONS

# ENGLISH

## Orthodontic Brackets with Pre-Applied Adhesive

The products covered by this instruction include:

- ClearVu® with Pre-Coated Adhesive
- InVu® with Pre-Coated Adhesive
- Tip-Edge® PLUS Ceramic with Pre-Coated Adhesive
- Nu-Edge NX® with Pre-Applied Adhesive
- Tip-Edge NX with Pre-Applied Adhesive

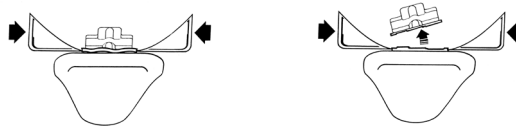*Apply slow, steady compressive force to the mesiodistal edges of the bracket's mesh base, making sure to be close to the enamel. The base will deform and debond.*

If excessive bonding adhesive prevents applying pressure to the base or tends to deflect the beaks of the ligature cutting plier away from the polymer mesh base, remove the adhesive from the mesiodistal surfaces of the base with a steel fissure bur. Avoid using a carbide bur or diamond instrument as they are not necessary and could accidentally mar the enamel surface. Once the adhesive is removed, apply compressive pressure directly to the edges of the bracket base near the enamel.

### • Metal Brackets

No special debonding technique is required for metal brackets with pre-applied adhesive. Remove the bracket using a debonding plier or instrument of doctor's preference. Apply gentle pressure until the bracket debonds.

## REBONDING

In the case of a premature debond of a bracket, do not rebond. The base would become damaged when debonding and would not bond to the enamel properly if reused. Clean excess adhesive from tooth, prepare the tooth surface according to the adhesive manufacturer's instructions, and bond a new bracket (with the same part number).

## SPECIAL CARE

- Do not touch bracket base prior to bonding.
- Remove any stray adhesive from bracket surface prior to curing. Cured adhesive covering any bracket identification or positional marking will prevent removal of these markings.

### Ceramic Brackets:

- Ceramic is such a hard material that abrasion of the enamel can occur if incisal surfaces occlude. Each patient's occlusion should be taken into consideration when determining if and when ceramics can be safely bonded and still avoid contact with opposing teeth.
- Do not force auxiliaries or archwires into ceramic brackets.
- Steel ligatures should not be placed as tightly on ceramic brackets as on stainless steel brackets.

## Storage Condition

Store brackets at recommended temperature (between 35°F and 80°F / 2°C and 27°C) for maximum shelf life.

## Shelf Life

30 months.

## REPORTING SERIOUS INCIDENTS

User and Patient Notice: If a serious incident occurs with this medical device, report it to the manufacturer and to the competent medical authority for the user/patient's country.

### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

## Orthodontic Brackets with Pre-Applied Adhesive

The products covered by this instruction include:

- ClearVu® with Pre-Coated Adhesive
- InVu® with Pre-Coated Adhesive
- Tip-Edge® PLUS Ceramic with Pre-Coated Adhesive
- Nu-Edge NX® with Pre-Applied Adhesive
- Tip-Edge NX with Pre-Applied Adhesive

[tportho.com](http://tportho.com)

CE 0297 Advena Ltd.

Tower Business Centre, 2nd Flr.  
Tower Street, Swatar, BKR 4013 Malta

ClearVu, InVu, Nu-Edge NX, Tip-Edge, and TPO are registered trademarks of TP Orthodontics, Inc. Patents apply.

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