INSTRUCTIONS

Pre-Finisher® Semi-custom Finishing Appliance — Size Selection

Click the language you wish to view for the Pre-Finisher Appliance to be taken directly to that page.

- English
- Dutch
- French
- German
- Italian
- Portuguese
- Spanish
- Chinese
- Japanese

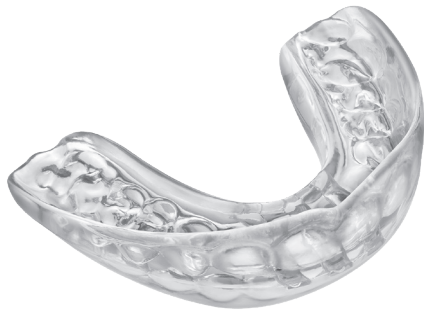
TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672
USA

tportho.com

Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

Pre-Finisher® Semi-custom Finishing Appliance Size Selection

Symbols Used on Labeling

	Manufacturer.
	Date of manufacture.
	Authorized Representative in the European Community.
	Batch code.
	Caution! See warnings and precautions.
	Consult instructions for use.
	Do not re-use.
	Use by date.
	Do not use if package is damaged.
	Medical device.
	Prescription device.

 TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672
USA

tportho.com

Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

Indications for Use

This appliance is intended for use in orthodontic treatment.

Warnings

- This is a single-use device, intended for use for only one patient. Any reuse of this product will run the risk of cross-contamination to patients.
- A special millimeter ruler is enclosed for every Pre-Finisher in this kit. Each ruler is designed to be used just once if placed in the mouth.

Precautions

The Pre-Finisher Appliance will fit most occlusions, either following the removal of fixed appliances at the end of aligner treatment or in nearly normal occlusions, including those that have minor orthodontic relapse. Exceptions would include:

- Cases with tooth mass discrepancies (one or more teeth that are relatively large or small)
- Cases which exhibit unbalanced extractions (only one tooth removed in one or both arches)

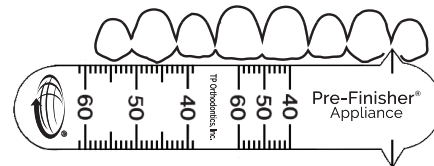
Contraindications

TP Orthodontics sells its products to trained dental professionals and orthodontists, who have the primary responsibility to identify any possible contraindications that may preclude the use of this product.

Instructions for Use

This product must be used by qualified orthodontists or dentists trained in orthodontics.

First, choose the appropriate appliance type from the following series: Non-Extraction, Four-Bicuspid Extraction, Asian Non-Extraction or Asian Four-Bicuspid Extraction.

Use one of the enclosed millimeter rulers to measure for the correct size. The ruler may be used on the right or left side, according to operator preference. A reading may be obtained one of two ways:

1. For quick, easy measurement, use the condensed scale in the center of the ruler. Place the point distal to either the right or left upper cuspid. Curve the ruler along the incisal edges. Reading at midline indicates total mesiodistal width of upper anterior teeth.

Note: Each mark on the condensed scale equals two millimeters.

2. For more accurate measurement, use the full-length scale at the end of the ruler. Place the point distal to the upper right or left cuspid. Bend the ruler along the incisal edges. Take reading distally to the opposite cuspid.

Once you have measured, select an appliance with a part number ending in two digits that match the millimeter measurement. If there are band spaces, select a Pre-Finisher one size (one mm) smaller.

Fit the Pre-Finisher onto the upper teeth and have the patient close firmly into the appliance. The cuspids should be tightly in contact with the distal surfaces of their respective sockets.

Clearly explaining the proper use of the Pre-Finisher Appliance to patients can often mean the difference between treatment success or failure.

Pre-Finisher®
Semi-custom
Finishing Appliance
Size Selection

Typical Wearing Schedule

First 2 days	Full-time wear (wearing appliance as much as possible)
Day 3 until follow-up	4 hours of daytime exercise wear* and overnight wear
First follow-up: 2 weeks after appliance delivery	If results are satisfactory, evaluate to discontinue wear of the appliance or reduce to 3 hours daytime (exercise wear*) and overnight
Second follow-up: 4 weeks after appliance delivery	Discontinue wear of the Pre-Finisher and deliver retainers

* Exercise wearing means that the patient should close the teeth into the Pre-Finisher Appliance until muscle fatigue occurs then relaxing with the teeth seated in the appliance.

If after the initial wearing period no changes have occurred, the Pre-Finisher has either not been worn enough, not been worn properly, or was improperly fitted.

Storage Condition

Clean, dust-free, cool, dry area.

REPORTING SERIOUS INCIDENTS

User and Patient Notice: If a serious incident occurs with this medical device, report it to the manufacturer and to the competent medical authority for the user/patient's country.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

Pre-Finisher® Semi-custom Finishing Appliance Size Selection

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