INSTRUCTIONS

TPO® Surface Conditioners for Bonding

Click the language you wish to view for TPO Surface Conditioners to be taken directly to that page.

- English
- Dutch
- French
- German
- Italian
- Portuguese
- Spanish
- Chinese
- Japanese

TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672
USA

tportho.com



0297 Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

TPO® Surface Conditioners for Bonding

The products covered by this instruction include:

- TPO Blue Etchant Gel
- Enamel Conditioner Solution and Gel
- Porcelain Primer

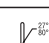Symbols Used on Labeling

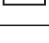	Manufacturer.
	Date of manufacture.
	Authorized Representative in the European Community.
	Batch code.
	Caution! See warnings and precautions.
	Consult instructions for use.
	Use by date.
	Keep away from sunlight.
	Store between 2°C/35°F (lower limit) and 27°C/80°F (higher limit).
	Do not use if package is damaged.
	Medical device.
	Prescription device.

 TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672
USA

tportho.com

 0297  Advena Ltd.
Tower Business Centre, 2nd Flr.
Tower Street, Swatar, BKR 4013 Malta

Indications for Use

Enamel conditioner and etchant are indicated for etching enamel and dentin to promote adhesion of primer/bonding agent adhesives to tooth structure and restorative materials.

Warnings and Precautions

- Best temperature for use is 68°-77°F (20°-25°C). Storage temperature is best at 35°-80°F (2°-27°C).
- Do not use after expiration date.
- Avoid contact with eyes, skin and soft tissues. In case of contact, immediately wash with water and get medical attention for eyes. Do not take internally. Wash hands after use. Discontinue use if irritation occurs.
- Porcelain Primer: Solution is flammable.

Contraindications

- Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.
- TP Orthodontics sells its products to trained dental professionals and orthodontists, who have the primary responsibility to identify any possible contraindications that may preclude the use of this product.
- Federal law restricts this device to sale by or on the order of a dentist.

Instructions for Use

This product must be used by qualified orthodontists or dentists trained in orthodontics.

ALL TPO SURFACE CONDITIONERS

1. Thoroughly clean teeth and surfaces to be bonded with plain pumice (no oils or fluoride) using a rotating bristle brush or rubber cup. Do not irritate the gingival tissue, as this may cause slight bleeding which could contaminate the bonding area.
2. Rinse teeth thoroughly with water, isolate teeth and air dry.

ENAMEL CONDITIONER AND ETCHANT

3. Apply directly to each tooth. Condition for 30 seconds. Do not condition area larger than required for bonding. All teeth to be bonded in one arch may be conditioned simultaneously.

Etchant syringe: remove cap from syringe and insert a syringe tip. After use, remove syringe tip and replace cap. Additional disposable tips may be ordered in packs of 25.

Enamel Conditioner: dispense onto work pad and, using cotton forceps and one foam pellet applicator, dab conditioner onto teeth to be bonded.

4. Flush teeth thoroughly with an oil-free air/water spray to remove all traces of conditioner and evacuate. Do not allow the patient to rinse or permit contact of the conditioned enamel with saliva or mucosal tissue. Re-isolate and dry teeth to be bonded with oil- and water-free air (preferably warm air). If conditioned areas do not appear chalky white, re-condition for approximately 10-15 more seconds.

Note: Enamel Conditioner Solution: if a portion of a tooth exhibits a greenish hue, dental plaque is still present and the tooth must be re-pumiced and re-conditioned.

INSTRUCTIONS

TPO® Surface Conditioners for Bonding

The products covered by this instruction include:

- TPO Blue Etchant Gel
- Enamel Conditioner Solution and Gel
- Porcelain Primer

ENGLISH

PORCELAIN PRIMER

5. Apply porcelain primer to the porcelain surface with a cotton or foam pellet. Allow the primer to remain, undisturbed, on the porcelain surface for 2 minutes.
Do not rinse to remove primer.

6. The porcelain surface is now prepared for bonding.

Note: In most cases, the above procedures will provide a secure bond of attachment to porcelain. If the bond fails, it will be necessary to break the glaze on the porcelain with a small stone and repeat steps 1, 2 and 5.

7. Removal of Attachments: apply de-bonding forces carefully to avoid damage to the porcelain.

Storage Condition

Storage temperature is best at 35°-80°F (2°-27°C).

Shelf Life

3 years for Enamel Conditioner Gel.

5 years for TPO Blue Etchant Gel in Syringe, Enamel Conditioner Solution and Porcelain Primer.

REPORTING SERIOUS INCIDENTS

User and Patient Notice: If a serious incident occurs with this medical device, report it to the manufacturer and to the competent medical authority for the user/patient's country.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

TPO® Surface Conditioners for Bonding

The products covered by this instruction include:

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